



1. Introduction

The heat-tightening instrument is an innovative blend of healthcare and medical beauty features that uses the latest, highest-end, 360-degree stereo radiofrequency to fully and uniformly heat the tissues around the vagina, directly transfer heat energy to the SMAS layer and stimulate collagen production, so that the vagina getting shrinking, compact and shaping effect. Convenient and efficient!

The instrument has 2 therapeutic probes, the inner treatment probe: noninvasive vaginal tightening, vaginal moisture smooth, restore the elasticity;

Vulva treatment probe: to improve the appearance of privacy, tighten the labia, pigmentation corrector, restore private part .

Private thermal tightening instrument to narrow the vagina painless, no wound, no recovery, no bleeding, no need for multiple treatments across the shrinking Shaping equipment.

2. Working principle

RF private tightening instrument with its unique heat energy transfer directly to the vaginal SMAS layer, enhance the SMAS fascia, a comprehensive deep solution to vaginal relaxation problem, the heat energy is accurately located in submucosal 4mm fascia layer, the fascia play Stretch muscle growth, to achieve the best effect of contraction of the vagina. At the same time acting on the subcutaneous 3mm collagen layer, collagen reborn, to the skin to restore elasticity, at the same time due to heat energy skimming the epidermis, so do not worry about mucosal injury, mucosal injury will not cause infection problems. The instrument through the operator will be vulva probe in the vulva operation can improve the appearance of the vulva, vaginal contraction, shaping a shy pink girl shy private parts. Re-use of the treatment of the head of the genital innervation of the vagina to tighten the vagina, contraction, to achieve the best moisture vaginal effect.

3. Technical advantages

1. By automatically controlling the rotation angle to ensure that multiple shoots are very precise, set corresponding parameters according to the vaginal depth, such as angle, time, energy. Greatly shorten the operating time, so that vaginal muscles get more uniform energy, better efficacy.
2. Use the most advanced high technology, according to the condition of the vagina with 2 handles- vulva and inner treatment handle. Energy slightly over the SMAS layer, 100% without any damage, painless and comfortable.
3. Dermal collagen and collagen fibers have thermal effects, while the fat layer and fascia (SMAS) thermal stimulation.
4. Easy to operate and convenient
5. 10 inches display.
6. Immediate tightening, firming effect after treatment
7. Can be restored right after treatment , does not affect the normal sex life.

4. Application

1. The one who dissatisfaction with private parts
2. Vaginal relaxation
3. Vaginal dryness
4. Sex apathy
5. Urinary incontinence
6. Pigmentation

5.Parameters

Output power	180W	Input power	110V-220V
Frequency	5MHZ	Current	10A
Probes working mode	Time is adjustable , rotating 360°	Treatment handles	2 pcs
Treatment probes	Vaginal treatment probe and Vulva treatment probe each 5 pcs		
Angel	5° - 360°	Energy	0-15
Other			
Environmental conditions	Temperature: 0 °C to 40 °C Humidity: less than 75% Air quality: non-corrosive particles, metal ions and dust particles in the air		

6. Instrument Structure

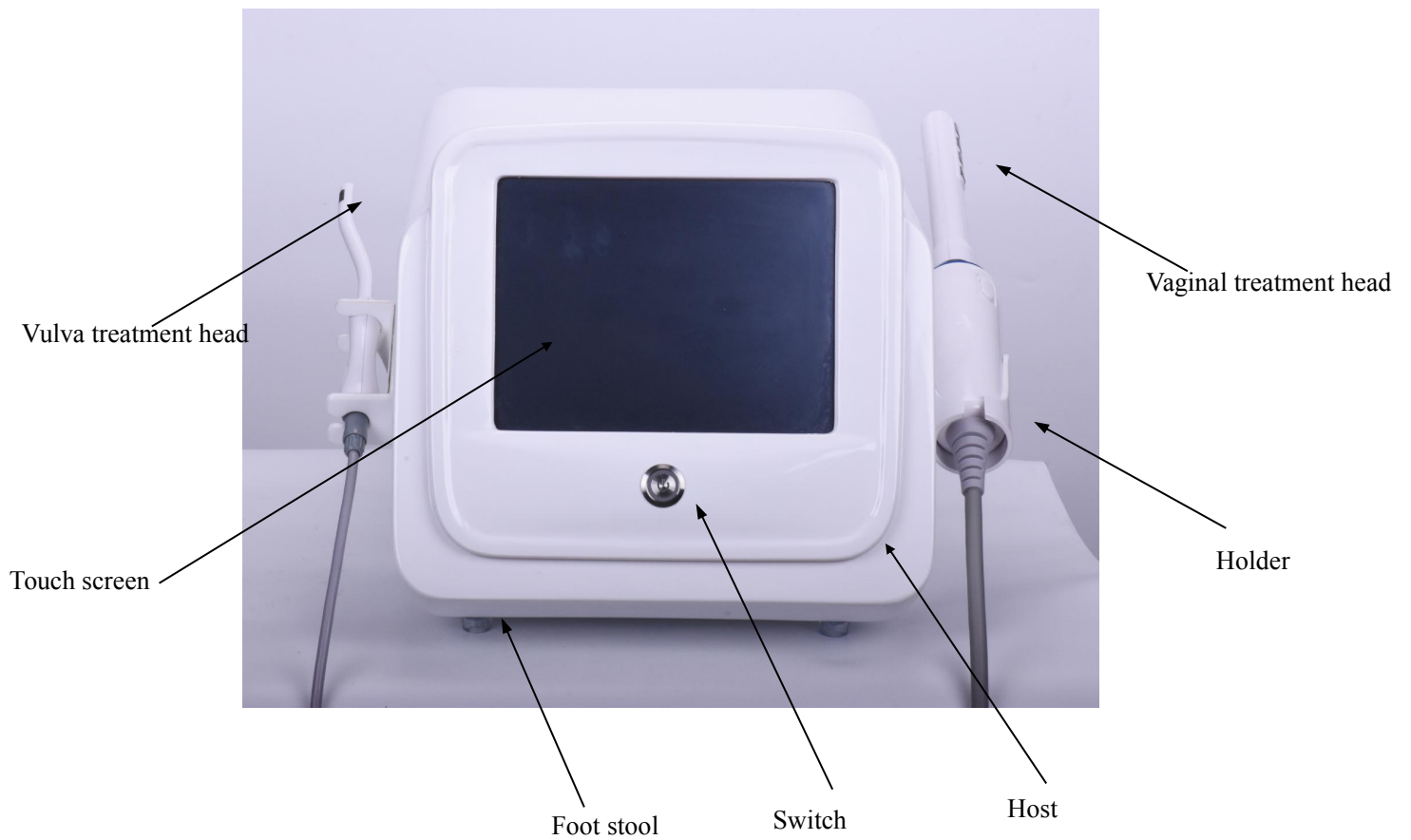


Fig. 1

Mainframe

The heart of machine, which includes the following parts:

- a) Power and control module: power to other modules, cut off the power in case of system failure;
- b) Control module: to control the coordination of other modules of the entire system;
- c) Display controller: display a variety of device information, and accept user instructions;
- d) High-voltage power supply: to charge the capacitor;

Treatment handle

1. Treatment head contains heat conduction of electro mechanical devices. It is connected to the system via a combination cable, with 4 thermally conductive sheets.

2. Press the switch on the treatment head to conduct heat, pay attention to metal cleaning to protect the thermal conductivity is very important. The treatment head is equipped with a thermo electric semiconductor to conduct thermal energy.

User operates the unit through the control panel which includes the following sections:

1, Power switch: red square switch, when the power is turned green. (Used to power off the unit urgently, press this switch will immediately disconnect the entire system power).

2, Display: User communicate with the system through the LCD screen and the touch key to . This is a 10 "LCD (liquid crystal display), the system status, user interface, tips and instructions to the user are displayed above.

3, Display switch: press this button screen is turned on (switch turn into green light)

4, START (start key): Click this button, the instrument can enter the working state.

5, STOP (stop key), click the button system to stop working.

6, Time (Time key), refers to the energy emission time, click the "-" time decreases, "+" time increases, adjustment range 0.1 ~ 3

7, Power (energy key): Click this key, click "-" energy is reduced, "+" energy is

increased, adjustment range 0 ~ 15

8, Back (Back key): Click to return to the previous interface.


Note:

Without the manufacturer's intention, the user can not disassemble the machine.

7. Interface


(1) Home page

Turn on power switch and screen switch , interface as below . Press one of the icon to choose treatment area after 3 minutes

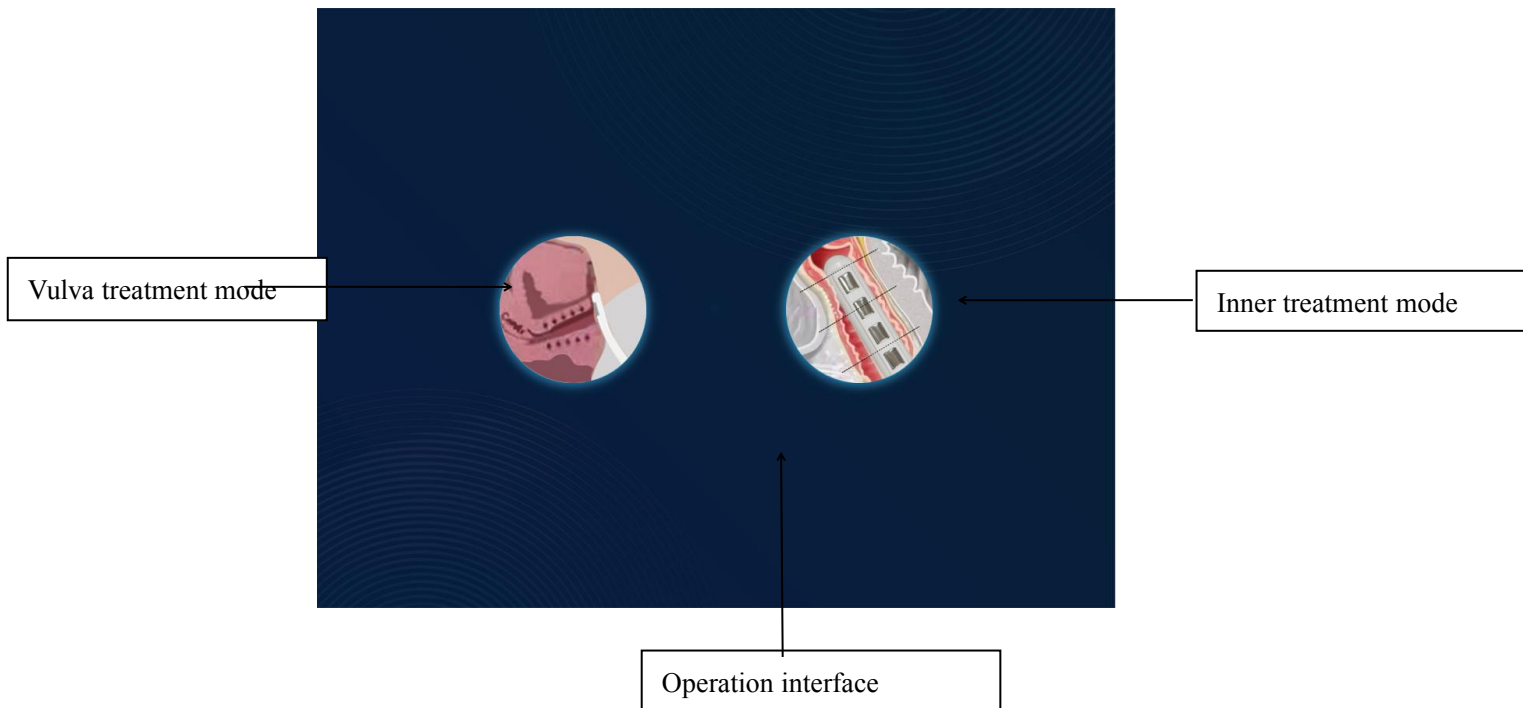

(FIG.2)

(2). Vulva treatment interface

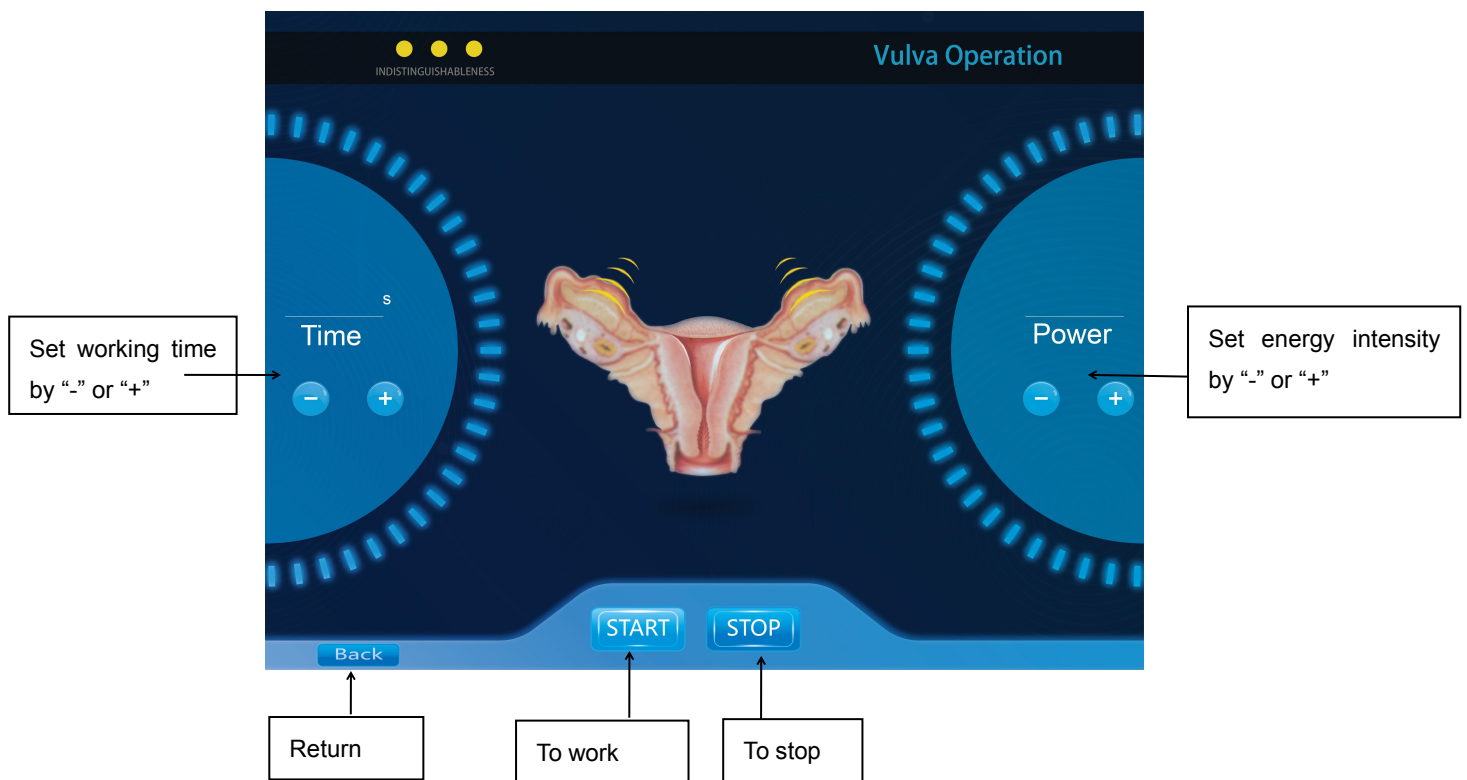

(Fig. 3)


After entering the vulva mode selection interface, as shown in Fig 3. Press "START" button to start working status, press "STOP" button to stop working and enter standby mode.

Time setting: 0.1~3S

Energy setting: 0~15 level

Back: Go back to last interface

(2) Inner treatment interface

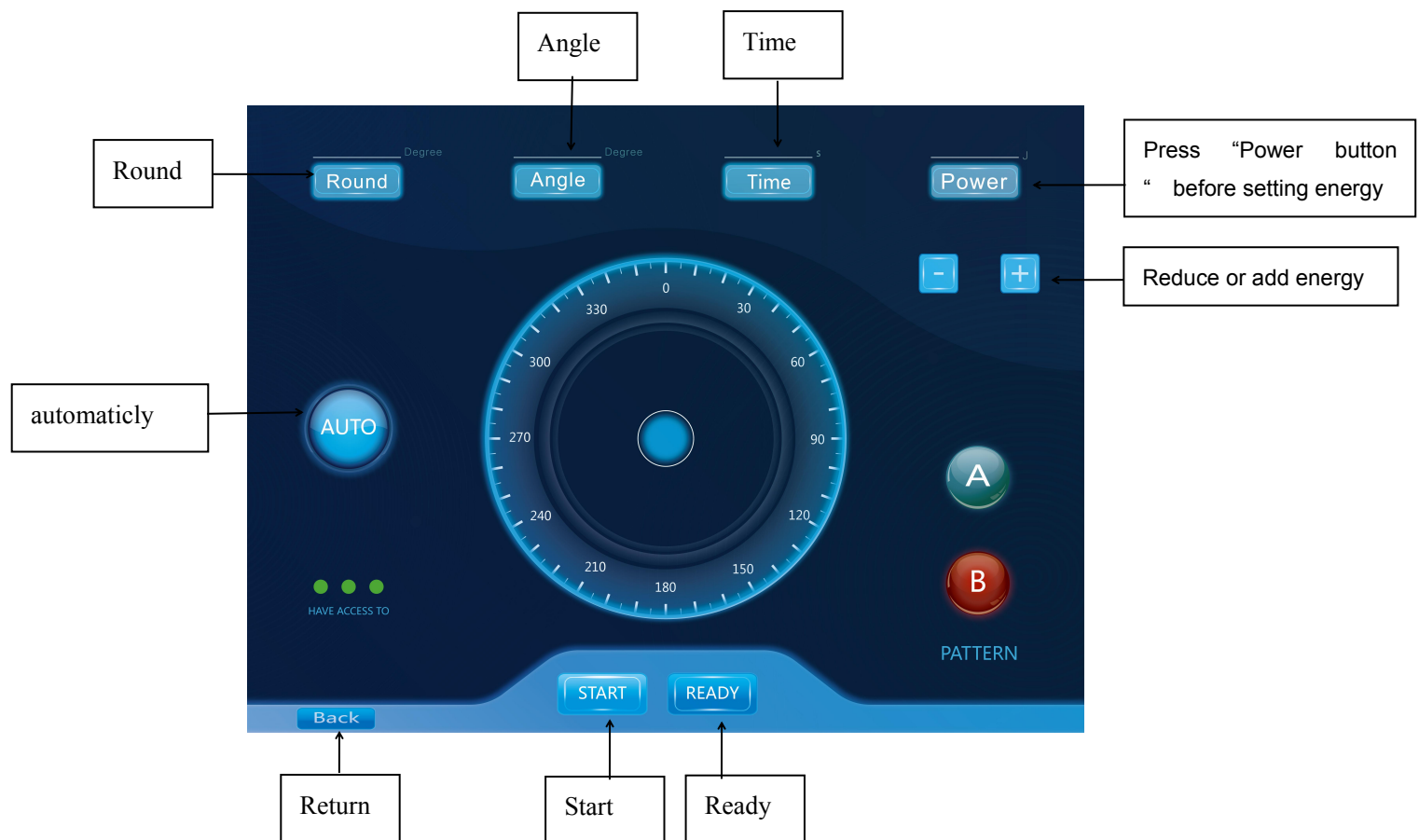

(Fig 4)

After entering the genital selection interface, as shown in Fig 4. Press the "READY" button to make the instrument system automatically rotate ready, press the "START" button to start working, into the working state.

Auto key: Click "AUTO" system to enter auto mode.

Round key: Click "Round" adjust the size of the angle value, each adjustment is 10, the adjustment range of 20 ° ~ 360 °

Angle key: Click "Angle" to adjust the angle value, adjust the range of 3 ° ~ 50 °

Time key: adjust the value of 0.1 ~ 5S

Energy key: Click "Power" to adjust the energy value. The adjustment range of 1 to 15

Regulate treatment parameters

In the treatment interface, we can adjust the treatment parameters. Click "parameters" interface to modify the parameters, the corresponding parameters will be selected, press "+" value increases, press "-" value decreases.

8. Operation introduction

A. Pre-operation preparation:

- 1) Install treatment head on the handle.
- 2) Ready for tightening gel/oil or private care solution
- 3) Cover the inner treatment head with condom. And apply the right amount of gel/oil.

B. Operation process

- 1) The gel evenly applied to the operation site, according to the customer's feeling appropriate to increase the amount of gel.
- 2) Press switch, the screen displays the home page interface, click the corresponding icon, set parameters and then press the start button to enter the work.
- 3) To feel the energy with your operator's hand first, then ask client touch the handle to feel whether the energy acceptable ,nomarily , patient should should feel thermal heating pass skin , a sense of micro-thermal and RF energy penetration, if there is no feeling may be appropriate to raise the energy parameters (Note: If patient has a numbness, please recheck whether the whole probe is close to the skin or the energy parameters should be lowered.)
- 4) The operator should always ask the customer's feeling,change the energy parameters according to customer's feeling . Energy intensity should be within the limits of the customer.

C . Operation points:

- 1) During treatment process , the heating probe should completely close to skin evenly , can not have gaps or one part of the probes not touch skin .
- 2) The machine has " tick tick tick" sound , means it works normally .
- 3) Intensity of operation must be uniform, must be close to skin.
- 4) The vaginal and vulva are sensitive parts , client may have tingling sensation , especially feel hot in inner vaginal , this is a normal phenomenon, strong penetration of radio frequency in order to achieve good results .

- 5) If client feel too strong heating , we can reduce energy accordingly.
- 6) Before the operation, if the customer is too dry, should apply with lubrication, **or it** will bring discomfort or stinging .
- 7) After the operation , doctor should inform customers to pay attention to private health matters, and post-maintenance.
- 8) Each customer with different feeling to same intensity energy, we adjust energy according to each customer's tolerance , the greater the energy will be better, if the customer can not tolerance too much energy, in order to ensure the effect, you have to repeat the operation several times.



Warning

This machine has high pressure, be sure to ensure that all the panels have been properly covered;
Do not point the treatment head anywhere beyond the target area;
Emitting too strong heat energy to treatment area may cause thermal damage to the skin.

Taboo crowd:

- 1, Pregnant women are forbidden to use.
- 2, Hypertension, diabetes, heart disease, the current sensitive, pregnancy, blood disease patients are forbidden to use.
- 3, The one who installed heart rhythm, or severe skin disease on treatment area.
- 4, Cervical erosion, vaginitis, pelvic inflammatory disease and infectious gynecological diseases.
- 5, Can not operate on the damaged skin.
- 6, Hemorrhagic disease or trauma, dermatitis, skin disease.



55 2390 7756



ventas@derillasass.com
gamuroli@gmail.com



www.derillasass.com



AngelSkin



ANGELSKIN