



Deriffa Sass
SPA - MEDICAL

MONA ONE

IPL + E-Light+SHR SHR-950B Hair Removal System

ANGEL
SKIN



Introduction of SHR-950B



1

Lasting Hair Removal

the unwanted hair under arms, legs, lips, or other body parts.

2

Skin Rejuvenation

shrink pores, tighten skin, improve skin elasticity and glossiness.

3

Pigmentation Removal

removes various kinds of pigment such as freckles, chloasma, age spots, sunburn, etc.

4

Acne Clearance



Specifications

Laser source: SHR-950B Hair Removal System

Wave length:	HR: 650-950nm SR : 560-950nm
Wave length:	Optional:(VR 420-950nm PR: 530-950nm AR:500-950nm)
Fluence :	IPL&E-light: 10-50J/cm ² SHR: 1-10J/cm ²
Frequency:	IPL&E-light: ≤0.5HZ SHR: 1-8HZ SHR&SSR:1-2HZ
RF power:	30W
Input Power:	3000W
Power source:	pure sapphire
Lamp:	imported lamp
Spot Size:	10*40mm for(SR,VR,AR,PR) 15*50mm for HR
Contact cooling temperature:	Max -10°C
Cooling system :	Built in water cooling +semi conductor cooling +air cooling
LCD screen:	10.4 inch true color touch screen control
Weight:	30KG
Machine size:	345mm×520mm×1090mm
Package size:	570mm×660mm×1200mm

SHR-950B Working Theory

1

SHR-950B is a revolutionary technology for IPL Hair removal treatment. The traditional IPL treat your skin with high amounts of Joules (energy), SHR-690 shots multiple shots but at low amounts of Joules.

2

SHR mode gently heats the hair follicles. Clients feel only warmth and a slight tingling sensation, some even comparing it to a light massage.

3

The system also utilizes In-Motion technology, where the hand piece is always in motion over the skin.



SPECTRA+ IDPL

Super Dynamic Pulse Light System

CETARON

WHERE TECHNOLOGY MEETS BEAUTY

START

MAIN MENU

SPECTRA+ IDPL

IPL
Intense Pulse Light

E-LIGHT

SDR
Super Dynamic Rejuvenation System

HR

SR

Back

IPL Mode

Handle Type: HR Hair Removal

Skin Type

1 Fair	2
4	5

Pigmentation

None	Light
Medium Heavy	Heavy

Hair Type

Small	Medium	Large
-------	--------	-------

Back

Next

IPL Mode

SPECTRA+ IDPL

Handle Type: HR Hair Removal

Skin Type

1 Fair	2	3
4	5	6 Darkest

Pigmentation

None	Light	Medium
Medium Heavy	Heavy	

Small Medium Large

Next

E-Light Mode

SPECTRA+ IDPL

AR Acne Removal

Handle Type: SR

Counter

Sum

Energy

J/cm²

W

Parameters

- 1st Pulse Duration ms
- 2nd Pulse Duration ms
- 3rd Pulse Duration ms
- Pulse Delay ms

Frequency

Standby

Back

SDR System

SPECTRA+ IDPL

Super Dynamic Rejuvenation

Handle Type: SR

Counter

Sum

Energy

J/cm²

Pulse Duration ms

Frequency ms

Back

Complete SHR-950B System

Pulse Mode




Pulse Mode




Pulse Mode




IPL Mode

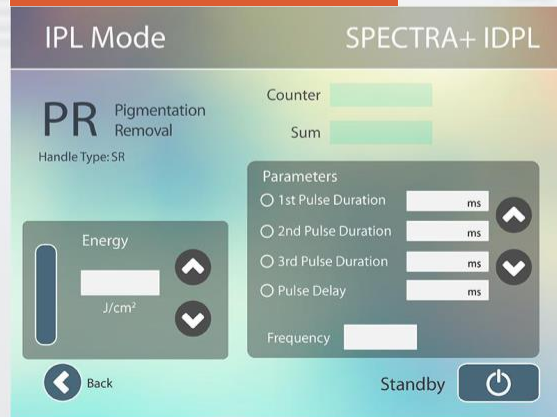

E-Light Mode

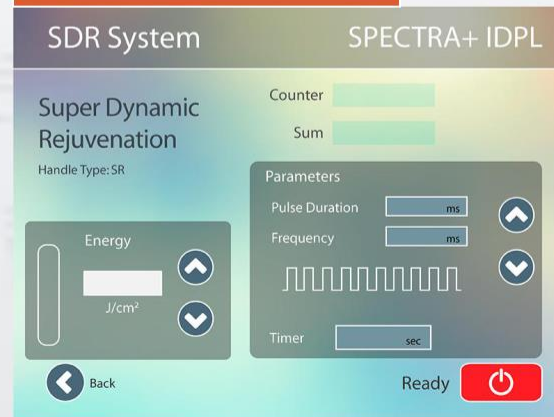

SHR Mode

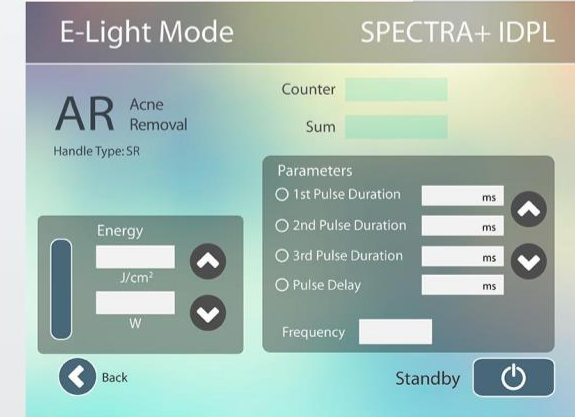

System Advantages and Feature

- 1 The system combines diode laser hair removal working modes includes SHR working mode to deliver multiple shots but at low amounts of energy, to achieve practically painless results.
- 2 The system combined with “In Motion” represents a breakthrough in permanent hair removal with light technology. The treatment is more pleasant than with conventional systems and skin is better protected
- 3 The system is suitable for all skin types, even tanners; Unlike traditional Laser and IPL, SHR can be had all year round without having to hide from the sun.
- 4 The system utilizes an imported flash lamp with a 300,000 shot life span.



System Advantages and Feature

5



The wavelength of 650-950nm for Hair removal and 560-950nm for skin rejuvenation are utilized, specially made crystal filters both violet light and infrared light which can eradiate and absorb large volumes of water from the skin.

6

SHR-950B is equipped with water temperature, water level, and water flow sensors to ensure safe operation. It will automatically stop the machine if there is any potential problem.

7

SHR 950B is a single pulse IPL machine. Only 4-5 treatments are needed, during which clients feel no pain.

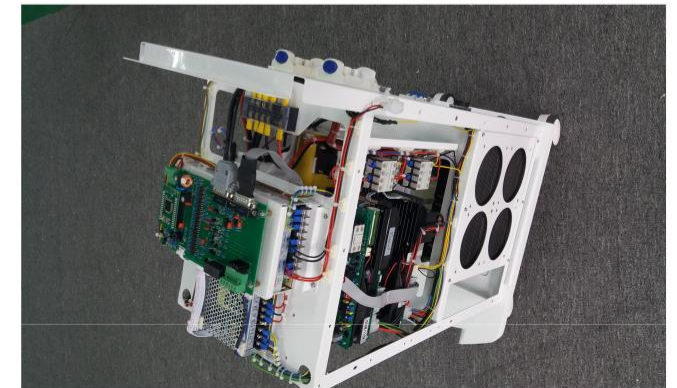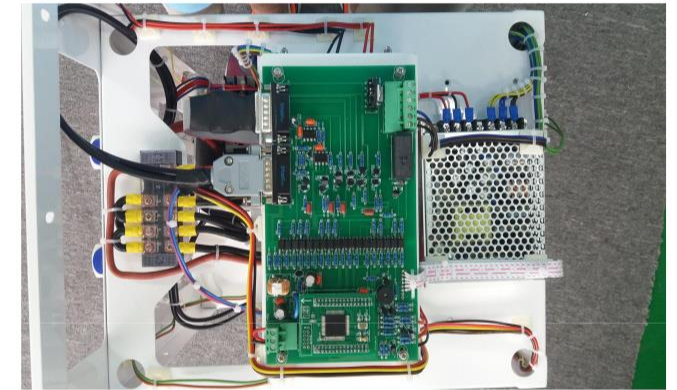

Difference	Traditional	SHR 950B	Advantage
Wavelength	650-1200nm (HR) 560-1200nm (SR)	650-950nm(HR) 560-950nm(SR)	SHR-950B filters both violet light and infrared light which can eradiate and absorb large volumes of water from the skin.
Auto identified hand piece	Unavailable	Available	Identified hand piece, HR &SR. The system can enter into correct operation menu directly when you insert the hand piece. Identified function makes the operation simple and treatment safe.
Frequency	0.3-1Hz	1-8Hz	Which speeds up the operation and reduces treatment time.
Work mode	Stationary mode	Stationary mode & Moving mode	Compared with traditional IPL treatment, only 1/8 as much time is needed for treatment of arms, legs, and back.
Water circuit monitor	Unavailable	Available	Water temperature, water level, and water flow sensors ensure the hand piece will not burn patients.
Stain resistant water circle	Unavailable	Available	Stain resistant water circuit system makes sure no components rust, keeping the crystal and filters clean and ensuring the equipment has a long lifespan.
Water lock connector	Unavailable	Available	Treatment head connector with water-lock for safe and easy changing, even with only single treatment head use is allowed.

Before and After Treatment



GRACIAS



55 2390 7756



ventas@derillasass.com
gamuroli@gmail.com



www.derillasass.com



AngelSkin



ANGELSKIN

